




Technical Data

Phonak TVLink II

The TVLink II basestation is a dedicated TV and audio interface for Phonak hearing aids that offers a great listening experience. The low latency transmission from TV to hearing aids in stereo provides excellent sound quality. The Phonak TVLink II basestation used with a Phonak streamer turns any wireless Phonak hearing aid into a lightweight wireless headset.

Description

- 1 Volume down
- 2 Power on/off and main indicator light
- 3 Volume up
- 4 Pairing
- 5 Power input
- 6 Digital audio input (optical)
- 7 Digital audio input (coaxial)
- 8 Analog audio input (3.5 mm)
- 9 Audio indicator lights
- 10 Replaceable charging slot

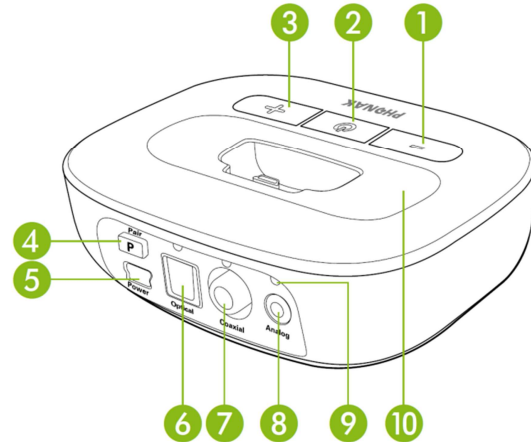

Special features

- Compatible with Phonak ComPilot , ComPilot II and ComPilot Air II
- Up to 30 meters (100 ft.) transmission range
- Exchangeable charging slot for streamers
- Complete set with power plug and cables
- No line of sight to hearing aids necessary
- Support of digital audio signals
- Automatic selection of input signal
- Test sound generation option
- Digital volume adjustment with audio-visual feedback

General information

Dimensions (L x W x H) :	101 x 77 x 30 mm (4.2 x 3.2 x 1.3 inches)
Weight:	100 g
Color:	Champagne
Operating conditions:	-10°C/55°C (14°F/131°F). Relative humidity of <95% (non-condensing)
Transport and storage conditions:	-20°C/60°C (-4°F/140°F). Relative humidity of 90% for a long period of time

Indicator light

Switched on:	 Solid green
Streaming:	 Solid blue
Pause mode:	 Blink blue
Pairing mode:	 Rapid blink blue
On but no audio input:	 Blink green
Test mode:	 Multi-color blinking




PHONAK

Power supply

Type:	DC, stabilized
Secondary voltage:	5 VDC
Max. current:	500 mA
Plug:	Mini-USB
Important notice:	Only use the original Phonak power supply or an USB certified power supply 5 VDC, > 500 mA

Audio information

Audio inputs:	<ul style="list-style-type: none"> Analog (3.5 mm plug) Digital optical (Toslink) Digital coaxial (SPDIF)
Priorities:	<ol style="list-style-type: none"> Optical Coaxial Analog 
Audio quality:	Stereo, 20 Hz – 22 kHz

Radio information

Bluetooth® profile:	A2DP, Advanced Audio Distribution Profile <ul style="list-style-type: none"> Optimized for use with Phonak streamers Compatibility with non-Phonak devices is not guaranteed
Frequency range:	2402.0 to 2480.0 MHz
RF power:	100 mW max. (10 mW max. for Japan/Korea version)
Transmission range to Phonak streamer:	Up to 30 m (100 ft.)
Bluetooth compliance:	Bluetooth version 3.0, Class 1
Max. paired devices:	4

Standards

Europe	EN 300 328 EN 301 489 EN 60950-1 IEC/EN 62311
Canada	RSS-210 Issue 8 RSS 102 Issue 4
Japan	ARIB-T66
USA	CFR 47, part 15 FCC OET Bulletin No.65




The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Phonak is under license. Other trademarks and trade names are those of their respective owners.

Compatibility note


Make sure the following combination is used:

Streamer:	ComPilot ComPilot II	ComPilot Air II
Charging slot:	Large	Small

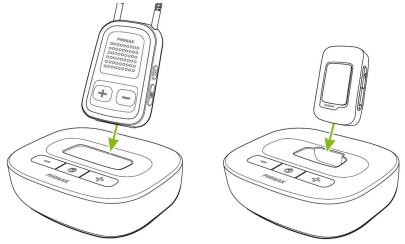


The stand-alone TVLink II basestation is delivered with the charging slot for the ComPilot Air II clip-on streamer. If you have ComPilot or ComPilot II, change the slot using the Phillips head screwdriver no 1.

Unscrew the bottom screw.
Lift the charging slot as shown.

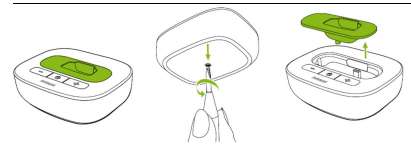


Insert the other slot completely.
Insert and gently tighten the screw.

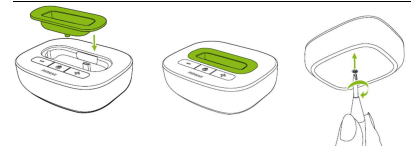

Bluetooth pairing information

TVLink II basestation

Switch on first.
Press the pairing button on the back.

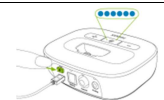

ComPilot and ComPilot II

Switch on first.
Press and hold the connect and volume up buttons at the same time for 2 sec.



ComPilot Air II

Switch on while pressing the volume up button at the same time.

